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Instructions for use

0506122016

Choose your language

- | | | | |
|-----------|-------------|-----------|-------------|
| PT | Português | SK | Slovensky |
| EN | English | PL | Polski |
| ES | Español | ET | Eesti |
| FR | Français | LT | Lietuviškai |
| IT | Italiano | LV | Latviski |
| RO | Română | CS | Česky |
| NL | Nederlands | GA | Gaeilge |
| DE | Deutsch | HU | Magyar |
| SV | Svenska | TR | Türkçe |
| DA | Danish | MT | Malti |
| FI | Suomi | BG | Български |
| HR | Hrvatski | RU | Русский |
| SL | Slovenščina | EL | Ελληνικα |

Português

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DEFINIÇÃO

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Hydrogen peroxide

DEFINITION

Tooth whitener for professional use based on 35% hydrogen peroxide.

INDICATIONS

Bleaching of discoloured teeth for professional use in the dental office.

COMPOSITION

35% hydrogen peroxide;

Thickener;

Desensitizing agents;

Glycol;

Neutralizing agent;

Preservatives;

Orange pigment;

Deionized water.

INSTRUCTIONS FOR USE

1. After cleaning the teeth, register the initial tooth colour using a VITA* shade guide or a photograph.
2. Dry the teeth and perform a careful relative

isolation with Clàriant Dam gingival barrier covering the gingival tissues which must be light cured for 10 seconds on each tooth.

4. Attach the two Clàriant Office syringes (A and B) together and mix their contents by alternately pushing the plunger up to 10 times, then push the entire mixed contents into one of the syringes;

5. Apply the whitening gel on the buccal and proximal surfaces of each tooth to be bleached. The gel layer should have a thickness of 1 mm;

6. The gel should remain on the teeth for up to 45 minutes. To release oxygen bubbles, use a disposable brush. If the patient reports any sensitivity, remove the product and establish the cause (cracks, infiltration, exposed dentin, etc.);

7. After 45 minutes, remove the gel with an aspirating endodontic tip (Flex Endodontic Aspirator) and rinse thoroughly with water;

8. Remove the gingival barrier and all residues with a dental probe. Apply the desensitizing agent Clàriant D-Sense for 10 minutes.

STORAGE

Store the product at temperatures between 5 and 25°C. Do not freeze the product or expose to high temperatures. Do not expose it to direct light.

PRECAUTIONS

Tooth whitening should always be carried out under the supervision of a dentist;

The patient must be in good oral and general health; This product is not recommended during pregnancy, breastfeeding and for children under 15;

The patient should be instructed on the technique and results of treatment;

The patient should be warned that Clàriant Office gel does not whiten dental restorations;

Tooth bleaching may be ineffective in cases of structurally altered teeth (amelogenesis, dentinogenesis imperfecta and fluorosis stains) or very pronounced discoloration caused by tetracycline and other drugs taken systemically during tooth formation;

The patient should be warned of a possible increase in tooth sensitivity to cold stimuli during treatment. To control this sensitivity, the application of Clàriant D-Sense desensitizing gel at the end of the bleaching session is recommended;

The 35% hydrogen peroxide content of the gel causes burning of the skin. All contact of the product with soft oral tissues, skin and eyes must be avoided. The dentist and dental assistants should wear gloves, masks and goggles. The patient should wear protective glasses and their oral soft tissues should be adequately isolated;

In case of accidental contact with the skin or intraoral tissue, neutralize the peroxide covering the area with a few drops of the Clàriant Neutralize solution and then wash;

This product cannot be used on patients allergic to any of its components;

This procedure can be repeated for a maximum of 4 sessions. In case of tooth sensitivity, the home

technique should be indicated (Clàriant Home). The sensitivity causes should be established. If these causes can be controlled (cracks, leakage, non-carious cervical lesions, etc), repeat the in-office procedure with Clàriant Office;

The whitening gel can be stored in the refrigerator after use for up to 7 days.

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The 35% hydrogen peroxide content of the gel causes burning of the skin. All contact of the product with soft oral tissues, skin and eyes must be

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35% hydrogenperoxid;
Fortykningsmiddel;

avoided. The dentist and dental assistants should wear gloves, masks and goggles. The patient should wear protective glasses and their oral soft tissues should be adequately isolated;

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This procedure can be repeated for a maximum of 4 sessions. In case of tooth sensitivity, the home technique should be indicated (Clàriant Home). The sensitivity causes should be established. If these causes can be controlled (cracks, leakage, non-carious cervical lesions, etc), repeat the in-office procedure with Clàriant Office;

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