Pt

## Fiberglass

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En

## Fiberglass

Fiberglass burs are efficient in removing excesses in the dental enamel, preserving its integrity.

**Indications:** Fiberglass is indicated for removal of remaining composites and/or adhesives after bracket fixation and/or removal from the dental enamel.

**Use:** Install the Fiberglass bur on the contra-angle. Apply it at low speed between 5,000 RPM and 10,000 RPM, under slight pressure and with water refrigeration. For removal of excesses of composite after fixation or removal of brackets, we recommend using a carbide multi-laminated bur followed by the Fiberglass bur.

**Shapes:** Available in 6 different formats.

**Composition:** Epoxy resin and fiberglass.

**Storage:** Store below 30°C, in a dry place, protected from light.

**Warnings, precautions and restrictions:** Professional use only. Fiberglass is not indicated for removal of resin-modified glass ionomer cement. When there is proven allergy to any of its components, the product must not be used. Follow the instructions of use for preventing excessive surface heating. The use in high speed may change the product features changing its performance. To avoid an accidental swallowing of this product it's necessary the application of a total isolation technique of the operator's field. Wear eye protection to shield against particles (Dentists and patients). Wear a surgical mask to avoid inhalation of particles. Use of a dental dam is highly recommended to avoid the potential ingestion of dental materials or other debris. Care should be taken to have good control on any rotary instrument to protect patients from injury. Contact with soft tissues (skin, gingiva, mucosa) may result in injury to the tissue. The use of applied suction techniques applied during the procedure help limit the problem of particles discharging while using rotary instrumentation. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Cleaning:** Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pure pH range between 6.5 - 7.5. Prepare the dilution solution according to the manufacturer's instructions. The water should be potable. For best effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the products and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the pieces with absorbent paper or disposable wipes. **Sterilization:** The burs should be autoclaved (126°C, 16 min) after each patient. **Reprocessing Limits:** Repeated processing can change the color of the product without compromising its performance. This is a reusable device. The manufacturer guarantees up to 10 reuses. Inspection: Carefully inspect each device to ensure that all debris has been removed. Observe damage / breakage / wear of the active part that prevents proper operation and do not use in this case.

**Connection with contra-angle:** Fiberglass is manufactured in compliance with specifications from the standard ISO 1797:2017.

Ref. Kit | Código / Code | Conteúdo / Contenido / Content  
 0.3100 | 0000020000 | 6 pontas de fibra de vidro / 6 fresas de fibra de vidro / 6 fiberglass burs

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Obelis S.A