Pt

Polimax

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En

Polimax

Impregnated felt disc for polishing.

Polimax discs are composed by felt impregnated with dehydrated abrasive paste, extra-fine aluminum oxide based, indicated for polishing of composites and dental enamel. The abrasive paste is activated with a few drops of water, providing excellent polishing and natural shiny. Versaplus mandrel system that works by pressure, easy and fast to set.

Use: Install the Polimax disc on the Versaplus mandrel. Make sure that the disc is fully seated on the mandrel. Insert the mandrel shank into the chuck of the contra angle handpiece. Perform a test run to check the effective rotary operation. Humidify the Polimax disc with a few drops of water to activate the abrasive paste. Apply the product in slow rotation, exerting low pressure with intermittent touches.

Warnings, precautions and restrictions: Professional use. Single use only. Its reuse may cause cross contamination. Polimax should be used in rotations between 4,000 and 10,000 rpm. The use of the mandrel with similar products is of the user's sole responsibility. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operative field. If there is proven allergy to any of its components, the product must not be used. Wear eye protection to shield against particles (Dentists and patients). wear a surgical mask to avoid inhalation of particles. Use of a dental dam is highly recommended to avoid the potential ingestion of dental materials or other debris.

Care should be taken to have good control on any rotary instrument to protect patients from injury. Contact with soft tissues (skin, gingiva, mucosa) may result in injury to the tissue. The use of applied suction techniques applied during the procedure help limit the problem of particles discharging while using rotatory instrumentation. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Connection with contra-angle: The Versaplus Mandrel is manufactured in compliance with specifications from the rule ISO 1797:2017.

Cleaning and sterilization: Mandrel - Cleaning: Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amylase, Protease, Lipase and Carbohydrazine) and with pure pH range between 6.5 - 7.5. Prepare the dilution solution according to the manufacturer's instructions. The water should be potable. For best effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the product and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the piece with absorbent paper or disposable wipes. Sterilization: The mandrel should be autoclaved (126°C, 16min), after each patient. Reprocessing Limits: Repeated processing has minimal effect on the mandrel due to its composition. This is a reusable device. The manufacturer guarantees up to 50 reuses. Inspection: Carefully inspect the mandrel to ensure that all debris has been removed. Observe damage / breakage of the product that would prevent proper operation and do not use in this case.

Composition: Discs: Aluminum oxide and synthetic wool felt. Fixing System: Polyvinyl chloride. Mandrel: Nickel plated metal. Adhesive: Structural acrylic adhesive.

Storage: Store below 30°C (86°F), in a dry place, protected from light.

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Obelis S.A.