Pulpo-San

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Pulpo-San

DENTISTRY CEMENT
LINER FOR TOOTH CAVITIES

Description

The product is made of a white powder, slightly grey and a clear liquid, slightly yellow, a predominant characteristic odour of eugenol, free of suspending particles or sediments.

Composition

Pulpo San Powder

Zinc Oxide	80,81 g%
Barium Sulphate	16,16 g%
Calcium Phosphate	2,02 g%
Zinc Acetate	1,01 g%

Pulpo San Liquid

Essence of clove (Eugenol)	95,00 g%
Thymol	5,00 g%

Properties

- Sets in 5 to 8 minutes.
- Has a regular friction resistance.
- Adheres to cavity walls.
- Shows weak solubility.
- Sets without producing heat.
- Easy to remove.

Indications

Pulpo-San is recommended for various applications, such as sedative bases, liners of cavities, dressing seals, temporary cementation of crowns, bridges, inlays and temporary fillings when removal are planned leaving the remainder to serve as an intermediate base under the permanent fillings.

Contra-Indications

Pulpo-San should not be used as a permanent fillings means.
It should not be used in case of proven allergy to the product.

Side Effects

When used in the correct way, it does not show side effects worth mentioning.

Interactions

The bottle containing Pulpo-San Liquid should be kept in a place separate and distant from the cements in general (silicate and glass ionomer) and resins, as this liquid contains mainly eugenol, the vapours of which can modify noticeably both the physical and

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INSTRUCTION FOR USE

Pulpo San can be prepared in the desired consistence, by just altering the proportions of powder and liquid.

The ideal consistence is better determined for each case, according to the work to be carried out.

To line cavities – after removing the tissue damaged by the caries, according to the technique in use, the cavity having been prepared and sterilised, the powder must be kept away from the liquid on the glass plate until the beginning of the spatulation. The recommended ratio is 170 mg of powder to 85 mg of liquid (ratio of 2 measures of powder for 1 of liquid). The spatulation is begun taking the powder to the liquid until they totally mix. The spatulation should be concluded in around 1 minute.

As a temporary cement – after removing the tissue damaged by the caries, according to the technique in use, the cavity having been prepared and sterilised, the powder must be kept away from the liquid on the glass plate until the beginning of the spatulation. The recommended ratio is 220 mg of powder for 73 mg of liquid (ratio of 3 measures of powder for 1 of liquid). The spatulation is begun taking the powder to the liquid until they totally mix. The spatulation should be concluded in around 1 minute.

WARNINGS

Before beginning cementing and the filling or lining, make sure that all the contact surfaces have been clean and dry.

Sterilise the cavity with a suitable germicide, dry it immediately with cotton and apply carefully the hot air jet.

PRECAUTIONS

- Do not use after expiry date.
- Keep out of children's reach.
- Dentistry use only.
- This material has been manufactured for dental use and it should be handled according to instructions.

STORAGE

In order to preserve the qualities of the product, it is recommended to keep it tightly closed in a fresh place, protected against heat and humidity and away from the light.

PRESERATION

- Pulpo-San powder – cartridge with 1 powder bottle (50g) and instructions for use.
- Pulpo-San liquid – cartridge with 1 liquid bottle (20ml), 1 liquid metering set.

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