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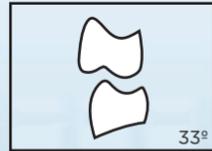
SYNTHETIC POLYMER TEETH FOR DENTAL PROSTHESES

Follow the usual procedures for choice, waxing-up, articulation, polymerization, infection-control and maintenance of the prosthesis, as well as for individual protection. For an optimized bonding to the denture base resin, grind the tooth base with a grinding instrument. After insulating the flask and prior to pack the acrylic dough, clean the base surface of the tooth using methylmethacrylate monomer.

Indications for the patient: To increase the teeth durability it is highly recommended to avoid contact with alkalis, acids or other substances that can damage acrylic materials during the daily hygiene.

Storage: Keep protected from light and heat.

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VIPI DENT Plus is an artificial acrylic tooth with high mechanical, chemical and wear resistance. They are very color stable, porosity absent and manufactured with biocompatible pigments. The moulds and shades are adequate to various ethnics and ages, with established and harmonized anatomy for *two layer teeth with complete facial harmony, offering an excellent aesthetic result.
*The VIPI DENT DECIDUOUS are one layer teeth.

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
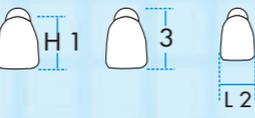
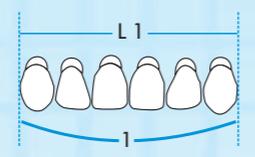
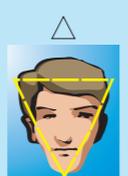
Formato del diente de acuerdo al biotipo
Tooth shape according to biotype

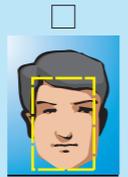Triangular Tapered
Esbelto Slender

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