VINTAGE Art Universal

- EN** Instructions for use
- DE** Gebrauchsanleitung
- FR** Mode d'emploi
- NL** Gebruiksaanwijzing
- ES** Instrucciones de uso
- IT** Istruzioni per l'uso

- RU** Инструкция по применению
- PL** Instrukcja obsługi
- RO** Instrucțiuni de utilizare
- SV** Bruksanvisning
- EL** Οδηγίες χρήσης
- PT** Instruções de uso

01:2019-04
71208

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EN Low fusing Stains and Glazes

READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

VINTAGE Art Universal is an universal stains and glaze assortment for shade adjustment of ceramic restorations including Zirconia and Lithium disilicate. The range of products has been coordinated with the layering, press and CAD ceramics from SHOFU INC. and can be used regardless of the CTE of the ceramic.

Color	Glass transition temperature	Coefficient of thermal expansion
GP, GP-F,	490 °C/914 °F	8,4 x 10 ⁻⁶ K ⁻¹
P, V, DR, Y, Gr, Bl, Bl-G, O, O-Br, Br, B, AS, BS, CS, DS, RS, LS, K	490 °C/914 °F	8,4 x 10 ⁻⁶ K ⁻¹
LY, LO, MP, Mlv, DR-Br, B-Br, Vn, G	490 °C/914 °F	8,0 x 10 ⁻⁶ K ⁻¹
W	490 °C/914 °F	7,5 x 10 ⁻⁶ K ⁻¹

VINTAGE Art Universal: Type 1 - Classe 1 (ISO 6872:2015)

INDICATIONS FOR USE

Shade adjustment of ceramic restorations

PRECAUTIONS AND WARNINGS

- If any inflammation or other allergic reactions occur on the operator, immediately discontinue use and seek medical advice.
- Avoid contact with skin and eyes. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- Wear protective glasses etc. while grinding and polishing this product to avoid damages to eyes.
- Use local dust extractor, dust protective mask, etc while grinding this product to avoid harmful influence of the dust on the human body.
- This product should not be used for patients with bruxism or occlusal abnormality etc.
- Follow the instructions for use of each material, instrument or equipment to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package or container.
(Example ☞ YYYY-MM-DD → Year-Month-Date of the expiration date)
- This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

- Cleaning of the restoration
Clean the restoration thoroughly with water or alcohol etc. to remove any dirt and other contaminants.
- Preparation
Dispense the necessary powder onto a glass slab or stain palette. The powders can be mixed at any ratio to obtain the desired shade. Add an adequate amount of VINTAGE Art Universal LIQUID to the dispensed powders and mix them together.
➤ The viscosity can be adjusted by adding some mixing liquid.
- Application
Apply the paste with a brush.
- Firing
Example of firing schedule

Drying temp.	Drying time	Vacuum-start temp.	Increasing temp.	Final temp.	Vacuum-end temp.	Hold time
400 °C 752 °F	6 min.	450 °C 842 °F	50 °C/min 90 °F/min	730 °C 1346 °F	720 °C 1328 °F	1 min

- Vacuum firing 1.3 – 8.0 kPa
VINTAGE Art Universal can be fired at 730 °C/1346 °F or higher temperature. If surface luster is insufficient, raise the final temperature.
- Optimal firing conditions vary due to the different designs and operating voltages of the porcelain furnaces. It is essential to carry out test firings before firing actual restorations.

NOTES ON USE

- Tightly close the cap immediately after each use.
- Dispense necessary amount of powder and mixing liquid. Do not put remaining material back into the container.
- Do not touch the material with bare hands.
- Do not use any liquid other than VINTAGE Art Universal LIQUID.
- Do not mix with any other products.

COMPOSITION

Alumino silicate glass, pigments and fluorescent material

STORAGE

Store at room temperature (1-30 °C/34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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