VOCO**CleanJoy®**

MD EU Medical device

Carefully read the instructions prior to use

Product description:

CleanJoy is a tooth cleaning and polishing paste used for professional tooth cleaning and polishing within the framework of a prophylactic treatment. The fluoride-containing paste (700 ppm fluoride) is offered in three different cleaning strengths, so that, depending on the degree of contamination, each patient can receive a treatment which addresses their individual situation. The safe application and easy identification of a particular paste with its own distinct level of cleaning is ensured by the colour coding of the tubes, *SingleDose* and pastes:

Red (RDA* = 195) = High cleaning strength
 Yellow (RDA* = 127) = Medium cleaning strength
 Green (RDA* = 16) = Polishing paste
 *RDA value as per DIN EN ISO 11609

Indications:

- Removal of surface, soft and hard plaque.
- Removal of extrinsic discolouration (e.g., staining from coffee, tea or tobacco).
- Polishing tooth surfaces within the framework of a professional tooth cleaning, prior to bleaching and following the removal of orthodontic devices.
- Cleaning and polishing restorations within the framework of a professional tooth cleaning.
- Removal of residues of temporary luting material prior to definitive bonding (cementation).

Contraindications:

CleanJoy contains artificial flavouring. **CleanJoy** should not be used in patients with a known hypersensitivity (allergy) to these constituents.

Patient target group:

CleanJoy is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

CleanJoy should only be used by a professionally trained dental practitioner.

Use:**Processing:**

- **Tube:** Remove the tube cap and dispense some of the material onto a mixing pad. **CleanJoy** can be applied with any commercially available cup or small brush. After use, replace the tube cap.
- **SingleDose:** Use the tab to remove the protective foil. Remove the material with a commercially available cup or small brush.
- In order to avoid potential heat generation and splashing, an appropriate rotational speed should be used when working with the paste (2,000 to 3,000 rpm).
- A final treatment with the polishing paste (green paste) must follow each and every application of a higher strength cleaning paste (red or yellow paste). The polishing paste can also be used following treatment with a powder jet device.
- After the application, rinse the cleaned surfaces thoroughly with water, remove remaining material by means of suction and ask the patient to rinse out their mouth thoroughly.
- Each *SingleDose* contains sufficient material for a single use. It must only be used once and only on a single patient. Repeated use can result in cross-contamination.

Warnings, precautionary measures:

- In case of contact with the eyes, rinse thoroughly with water and consult a doctor.
- An excessive intake of fluoride may cause nausea. Do not swallow **CleanJoy**!
- The information and/or advice we provide does not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Store at 39°F to 73°F (4°C to 23°C). Do not use **CleanJoy** after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **CleanJoy** must be reported to VOCO GmbH and the responsible authority.

PRODUCT ORDERING INFORMATION:

Tube 100 g fine mint	REF 2090
Tube 100 g medium mint	REF 2091
Tube 100 g coarse mint	REF 2092
Tube 100 g fine cherry	REF 6098
Tube 100 g medium cherry	REF 6099
Tube 100 g coarse cherry	REF 6100
Tube 100 g fine caramel	REF 6090
Tube 100 g medium caramel	REF 6091
Tube 100 g coarse caramel	REF 6092

<i>SingleDose</i> 200 x 2 g fine mint	REF 2096
<i>SingleDose</i> 200 x 2 g medium mint	REF 2097
<i>SingleDose</i> 200 x 2 g coarse mint	REF 2098
<i>SingleDose</i> 200 x 2 g fine cherry	REF 6102
<i>SingleDose</i> 200 x 2 g medium cherry	REF 6103
<i>SingleDose</i> 200 x 2 g coarse cherry	REF 6104
<i>SingleDose</i> 200 x 2 g fine caramel	REF 6094
<i>SingleDose</i> 200 x 2 g medium caramel	REF 6095
<i>SingleDose</i> 200 x 2 g coarse caramel	REF 6096

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.**For dental use only.**

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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