Whiteness HP

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Dental Whitening for Vital and Non-Vital Teeth

For Professional Use Only

Read the information in this manual carefully before using the product. Save it for reference at least until the product is completely used and/or until there is no more interaction between the product and the last patient.

Product Description

Whiteness HP is a dental whitening gel with a formula based on hydrogen peroxide at 35% recommended for professional use in the dental office only. The product is sold in two distinct phases: one called “thickener” and another called “hydrogen peroxide”. Before using the product, one phase must be mixed to the other in the right proportion, generating a viscous gel. The initial mixture has a reddish color that changes to colorless as the chemical reaction progresses allowing for the professional to identify exactly the areas where the gel has been applied. The product’s activation mechanism is based on the chemical reaction between the hydrogen peroxide and the pigments inside the dental structure (enamel and dentin), and that reaction is independent of the application of light sources (ex.: LED, laser, etc.).

Commercial Availability

Check commercial availability in your country.

Kit with gingival protector containing:

- 01 bottle with 10g of concentrated hydrogen peroxide;
- 01 bottle with 5g of thickener;
- 01 bottle with 2g of peroxide neutralizing solution;
- 01 spatula and 01 recipient for gel preparation;

- 01 Top dam with 2g and 6 tips.

Kit without gingival protector containing:

- 01 bottle with 10g of concentrated hydrogen peroxide;
- 01 bottle with 5g of thickener;
- 01 bottle with 2g of peroxide neutralizing solution;
- 01 spatula and 01 recipient for gel preparation.

Mini kit containing:

- 01 bottle with 4g of concentrated hydrogen peroxide;
- 01 bottle with 2g of thickener;
- 01 bottle with 2g of peroxide neutralizing solution;
- 01 spatula and 01 recipient for gel preparation.

Basic composition

After the mixing of the phases: hydrogen peroxide, thickener, red dye, glycol and inorganic load.

Product indications

The product is recommended for the following cases:

- Naturally dark or yellowish vital teeth.
- Vital teeth darkened by age.
- Vital teeth that are darkened or stained by extrinsic substances such as pigments from coffee, tobacco, tea, etc.
- Vital teeth with light to moderate stains due to fluorosis.
- Vital teeth stained by tetracycline, in case of light stains.
- In more challenging cases (severe darkening) the product can be combined with the at-home whitening technique.
- If the patient submitted to the treatment has non-

vital teeth, the product may be used externally or internally during the session in the clinic. Additionally, internal whiteners such as Whiteness Super-endo our Whiteness Perborato may be applied as intracoronary dressing after a precise clinical evaluation.

Precautions

- The product is for the exclusive use in dental office and must be handled only by professional dentists.
- The patient must be in good buccal and general health condition before beginning the treatment.
- Due to its high concentration, the gel should not get in touch with soft tissues. For that, the use of a gingival protector and a lip retractor is required.
- During product handling, both the professional and the assistant must wear protective gloves and goggles. The patient must also wear protective goggles and other necessary protections to avoid accidental contact of the product with the skin and clothes.
- The whitener is composed of a phase called peroxide (phase 1) and another phase called Thickener (phase 2), which must be mixed at the moment of application. The professionals must be careful when manipulating the peroxide phase because it contains hydrogen peroxide at 45-50%.
- The content of the bottle with peroxide may be under pressure and/or may have leaked due to adverse storage conditions. Before opening the package, protect hands and eyes (professional dentist and assistant) and make sure there is no liquid leaking in its interior. If there is a leak, request the replacement of the material.
- Handle the product away from the patient over a sink or a safe space and without risks of contaminating

other people.

- Whiteness HP is a whitener that is highly oxidizing and when in contact with live tissues may cause a white stain and temporary irritation (usually disappearing in 2 hours or less, without further complications). Avoid prolonged contact between the product and live tissues.
- In case of accidental contact with the skin or intraoral live tissues, inhibit the peroxide covering the area with a few drops of Neutralize solution and wash.
- After using the product, thoroughly wash hands and materials contaminated with the product.
- Patients must be oriented to avoid ingesting acid and/or highly colored foods for at least 24 hours after the whitening procedure to avoid sensitivity and not to compromise the treatment.

Contraindications

- It is not necessary to use accelerating light sources (LED, high intensity LASERS or others) since those light sources do not improve results neither reduce gel application time.
- Dental enamel acid etching before the whitening is not necessary or neither recommended.
- The product is not recommended for patients who have recently undergone gingival surgery or that have periodontal diseases, sensitive teeth, exposed roots, cavities, ulcers or bad health in general. In those cases, the solution of the problem must be provided before beginning the treatment.
- The product is not recommended for whitening teeth with amylogenesis and dentinogenesis imperfection, severe fluorosis, intense tetracycline stains and other anomalies of the enamel and dentin

that may compromise the vitality of the teeth.

- The product is not recommended for use in patients under anesthesia. It is important to monitor the sensitivity felt by the patient during the whitening procedure. In extreme cases in which there is an imperfection or flaw in the dental structure that allows for the peroxide to reach the dental pulp in greater amounts, pulp death may occur.
- The product is not recommended for use in pregnant or lactating women or in patients younger than 15.
- The product is not recommended for patients allergic to the components of the product.
- The product is not recommended for use with high-intensity lasers or other light sources that generate high temperatures on the dental surface. The temperature of the pulp should not exceed 42°C to avoid irreversible damage to its vitality.

Side effects

- Dental hypersensitivity may occur during or after whitening. Since this is hard to predict in several situations, the use of the desensitizer Desensibilize KF 0.2% - At-home use - (FGM) or Desensibilize KF 2% (FGM) (in-office use) is recommended for a prophylactic desensitization before the beginning of the treatment.
- Dental whitening may increase the visibility of pre-existing stains due to hypoplasia of the enamel (opaque white stains). In most cases, no additional care is necessary. The stains will disappear after the end of the treatment and the teeth will recover their natural hydration. In the cases of more resistant stains, treatment with fluoride in the area may help its disappearance.

Mode of use

Preparing for the whitening

1. Put on (dentist and assistant) your safety equipment (gloves, goggles, cap, mask and scrubs) to manipulate the product. Simultaneously, also, make sure your patient also wears protection to avoid accidental contact with the gel, using cap, goggles and an adequate lip retractor (Arcflex – FGM).
2. Remove the whitener from the package following the safety recommendations.
3. Vigorously shake the thickener flask to homogenize its content. Failure to do so may result in a low-viscosity gel (insufficient viscosity).
4. Prepare the recipient and the spatula for the mixing.

Note: if the whitener is being stored under refrigeration, take it out of the refrigerator at least 30 minutes before the beginning of the treatment for it to reach room temperature.

Whitening of vital teeth

Evaluate carefully the buccal cavity of the patient (read warnings and contraindications). The prophylaxis and desensitization with Desensibilize KF 2% (FGM) for 10 minutes before the procedure are recommended. This precaution aims at reducing the chances of dental hypersensitivity during the whitening treatment.

Step by step for vital teeth

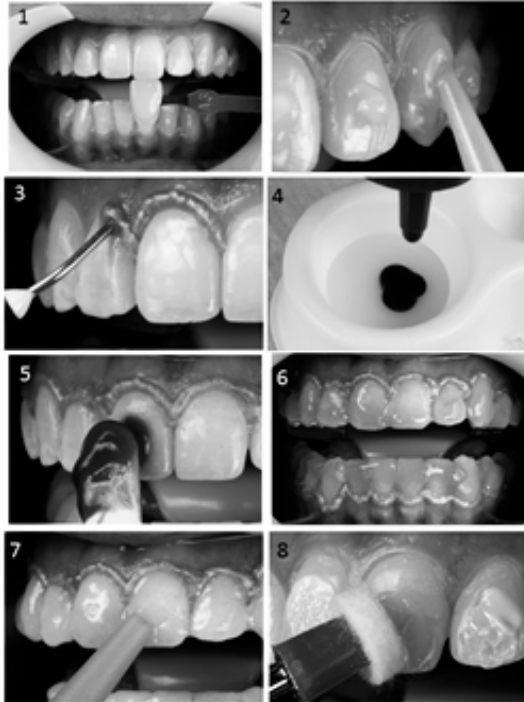
Fig 1. Select and register the shade of the patient's teeth with a shade guide and/or photograph before starting the whitening.

Fig 2. The application of Desensibilize KF 2% (FGM) for 10 minutes before the whitening is recommended in order to prevent possible dental hypersensitivity during and after the treatment. The choice for applying the desensitizer after the whitening is also valid, however, less efficient in certain cases. In cases of high hypersensitivity after the session, Desensibilize 0.2% (home desensitizer) may be applied for 10

minutes in an individual custom tray, by the patients themselves.

Fig 3. After recording the shade, insert the lip retractor Arcflex (FGM) to facilitate access to and handling of the buccal cavity. Promote relative isolation with the light-curing gingival protector Top dam (FGM) covering the marginal gum and the papillae with a layer of 3 to 5 mm of width and 1 mm of thickness. The protection must cover approximately 0.5 to 1 mm of the dental surface and of the gum to promote a good seal.

Use a clinical mirror looking from incisal to cervical and make sure there is no uncovered gingival tissue. If so, make the correction. This phase is crucial for avoiding contact of the peroxide with the gums. After the correct application, photopolymerize the Top dam composite (FGM) for 20 to 30 seconds for each group of 3 teeth. The gingival protector that is formed is rigid and insoluble preventing any possible irritation by the whitening agent.

Fig 4. Using the mixing recipient that comes with the kit, mix the peroxide phase (phase 1) with the thickener phase (phase 2) in the proportion of 3 drops of peroxide for 1 drop of thickener. The number of drops to be counted for each phase varies according to the size and number of teeth to be whitened. The 18:6 proportion (peroxide/thickener) is usually enough for the application on the superior and inferior arches from second pre-molar to second pre-molar. Shake the thickener bottle vigorously before using it.

Fig 5. With the help of a spatula or brush, completely cover the vestibular face of the teeth to be whitened with the whitening gel, including the interproximal and extend a little to the incisal and occlusal faces. The gel layer should be between 0.5 and 1mm thick.

Fig 6. Let the gel remain on the dental surface for 15 minutes. Using a Cavibrush brush or microapplicator (FGM), move the gel over the teeth three to four times to release any oxygen bubbles and promote the best possible contact of the gel with the teeth. At the end of the recommended time, remove the gel with an aspiration cannula (eg endodontic cannula) and clean with gauze to make the teeth ready to receive a new application of gel. Repeat steps 4 through 6 twice more in the same session for a total of 3 applications of 15 minutes each.

Fig 7. At the end of the treatment, remove the gel and wash the teeth. Remove the gingival protector detaching it with a probe.

Fig 8. Polish the teeth with Diamond Excel (FGM) polishing paste and Diamond (FGM) or Diamond Flex (FGM) felt disks.

Whitening non-vital teeth

Before starting the whitening process, get a radiography of the teeth to be whitened for checking its conditions. The treatment must be within acceptable endodontic standards.

Step by step for non-vital teeth

1. Select and record the shade of the patient's teeth by means of a shade guide or photograph before whitening.

2. Carry out an adequate coronary opening removing existing restorations, dentin with cavities, etc. Remove 3mm of restorative material from the root canal and proceed with the cervical sealing of approximately 2mm, with glass ionomer or composite (adhesive protocol), in order to prevent the diffusion of the whitening agent to the periodontal area.

3. Isolate the teeth with Top dam (FGM) (vestibular and palatine) and mix the phases of the whitener

Whiteness HP in the proportion 3 drops of phase 1 (peroxide) for 1 drop of phase 2 (thickener). Generally, 3 drops of peroxide and 1 drop of thickener are enough for the application in one tooth. Vigorously shake the bottle of the thickener before application.

4. Apply Whiteness HP on the vestibular face inside the pulp chamber. An approximately 1 mm thick layer is enough.

5. Remove the gel using aspiration (endodontic cannula) before reapplying the product. In a single session, up to three applications of the gel may be carried out. After the finalization of the applications of the whitening gel, wash and dry the tooth.

6. Remove the Top dam (FGM) and restore temporarily the cavity and follow up the result of the whitening after one week. If necessary, repeat the session for up to four times.

Note: *a non-vital tooth temporarily restored has a weakened crown due to the absence of the dental structure in its interior. Instruct the patient to be careful not to break the crown of the weakened tooth.*

7. As an option, after the removal of Whiteness HP, the professional may use the whitener Whiteness Super-endo (FGM) (carbamide peroxide at 37% for internal use) and continue the whitening of the tooth during the period between the sessions with Whiteness HP (for using the whitener Whiteness Super-endo, check its specific instructions).

At the end of the whitening procedure, restore the whitened tooth permanently. It is recommended to wait a minimum of 7 days before the restoration is made (time for the color to stabilize and for the elimination of residual oxygen).

Additional information

General:

- Most of dental stains may be removed with only one whitening session. However, depending on the origin and the age of the staining, there are cases in which more than one session may be necessary.
- If the whitening obtained in one session does not fulfill the expectations and the patient does not experience sensitivity or any other counter indication, the reapplication of the product may be carried out for up to two more sessions. In case more than one session is necessary, a minimum interval of 7 days between sessions should be observed.
- In case the patient develops sensitivity that cannot be controlled, the professional may alternatively decrease the whitening time and divide it between 3 sessions. In that case, it is suggested that the professional checks if there is no abnormality in the dental structure that may be causing the sensitivity (fissures, dentin exposure, etc.).
- When the hypersensitivity is difficult to control or, for any reason, a new whitening session is not possible or recommended, the whitening treatment may be complemented with whiteners for at-home use (Whiteness Perfect carbamide peroxide at 10%, for example).
- Like any other whitening technique, the shade record of the teeth before treatment, the warnings to the patient about the real perspectives of his/her case (limitations in cases of tetracycline stains, greyish color, etc.), warnings about possible sensitivity, need to replace restorations, are recommended. There are cases in which, due to particular characteristics of the teeth (enamel, type

of pigmentation, etc.) the whitening level desired may not be possible.

About the gel:

- Soon after manipulation, the product shows a carmine red color and that color changes to colorless within a certain period. That process of color transition generally takes 15 minutes. The color change of the gel is not an indication of the extinction of the activity of the hydrogen peroxide. The product has a high concentration of hydrogen peroxide even after the color change.
- Store the containers vertically to prevent the increase of the internal pressure from causing leaks.
- A certain variation in the concentration of hydrogen peroxide may be expected as a result of variations in the size of the drops during the preparation of the gel. To minimize that variation, be careful about the drops not to contain air bubbles to keep the drops a consistent.

Conservation and Storage:

Close bottles well avoiding external residue and keep them in their original package or in a safe place, in the vertical position out of the reach of non-trained personnel.

Store the product under temperatures between 5°C and 25°C / 41°F and 77°F.

Do not freeze the product.

Keep away from direct sunlight.

Handling instructions

Wash the gel preparation recipient with abundant water, use paper towels to dry and store it in the kit package.

Do not use heat or autoclave to dry or sterilize the gel preparation recipient or the spatula.

Those pieces cannot stand high temperatures.

For the disposal of the product, follow the disposal legislation of your country.

Warnings:

Do not use the product after its expiration date.

Keep out of the reach of children.

This material is for dental purposes only and should be handled and stored only in accordance with its instructions for use. The manufacturer will not be held responsible for any damage caused as a result of misuse, improper storage or improper handling. In addition, the user is required to check and verify, before handling and / or application of the product, and under their responsibility, whether this material is compatible with its intended use. Moreover, it is emphasized that data descriptions do not constitute any kind of guarantee and, therefore, have no binding.

Legendas de símbolos Leyendas de los símbolos Explanation of symbols	
	Data de fabricação Fecha de fabricación Date of manufacture
	Número do lote Número del lote Batch Number
	Prazo de validade Fecha de validez Shelf life
	Limites de temperatura Límites de temperatura Temperature limits
	Não utilizar se a embalagem estiver danificada No lo utilice si el empaque está dañado Do not use if packaging is damaged
	Conservar seco Consérvelo seco Keep it dry
	Manter afastado da luz solar Manténgalo lejos de la luz solar Keep protected from sunlight
	Fabricante Fabricante Manufacturer
	Consulte as instruções de utilização Consulte las instrucciones de utilización Refer to instructions for use
	Irritante Irritante Irritant