Presentation: 1 syringe with 2g, 10 tips and 1 instruction manual. Description: FIX LV is an orthodontic adhesive composed by methacrylate monomers and silica particles, providing a better adhesion on the surface of the tooth. A low-viscosity, photopolymerizable product, which does not drain, which allows the professional to dominate the setting time of the material and place the wire after attaching the brackets. Due to its hydrophobic features, it should be handled on dry teeth, without the contamination of the saliva or the oil of the jets, which may interfere with the adhesion on the enamel. The need for acid conditioning prior to its use is an essential factor to a successful result.

Composition: BisGMA, TEGDMA, UDMA, HEMA, photoinitiators, BHT, stabilizers, and charges.

Indication: FIX LV is a low-viscosity, photopolymerizable product, which promotes the placement of metal, ceramics and polycarbonate brackets by indirect bonding.

Directions of use:

Laboratory phase: **1.** Select the brackets; **2.** Select the trays; **3.** Perform the prophylaxis of the dental arches which will be cast; **4.** Perform the cast with alginic acid. Create the model with dental stone, which must not present any blisters or flaws; **5.** Perform the cutting of superior and inferior models so that the superior model could not include the palatal area and the inferior area could not include the buccal floor area. Both the models must not present any retentivity. Perform the cutting of superior and inferior models; **6.** Delineate the lines of orientation, through the vestibular surface and the lingual surface, corresponding to the long axis of the teeth, by marking also where the accessories (brackets and hooks) will be placed. Apply the isolation for resins, preventing the direct contact between the stone and the resin. Dry for 30 minutes; **7.** Bond the brackets and hooks with FIX LV at the demarcated level, removing the excess of resin. Photopolymerize each tooth for 40 seconds.

Preparation of the Tooth: **1.** Perform the prophylaxis of the teeth to be bonded, with the help of a Robson brush, pumice, and water, in order to remove the biofilm and other contaminant agents from the surface of the enamel; **2.** Wash the cleaned surfaces with water; **3.** The teeth isolation control is essential: place cheek retractors, isolating and drying the operation field with cotton rolls and keeping the high power saliva pump on; **4.** Perform the conditioning of the enamel, generally performed with a 37% phosphoric acid gel applied from 15 to 30 seconds. Vacuum the acid agent from the surface of the teeth; **5.** Rinse thoroughly with water the area on which the acid was applied, keeping the saliva pump in its position. Thus, the contamination of the enamel with the saliva is prevented. If such a thing happens, repeat the acid conditioning procedure by a few seconds (10 seconds maximum); **6.** Dry the entire surface of the enamel with air jets free from humidity or oil, keeping the operation field dry. The tooth enamel must have a white, flat, dull appearance; if such aspect is not observed, as observed commonly on very old enamels, the application of the phosphoric acid gel by more 10 seconds must be repeated; **7.** Apply a thin coat of adhesive on the teeth and on the individual bases of the brackets and photopolymerize the adhesive by 20 seconds on each tooth and each base; **8.** Place FIX LV on the base of all brackets, position the tray (one arch per time) by adapting them to the teeth and by pressing them gently; **9.** Photopolymerize each tooth for 40 seconds; **10.** Remove the tray carefully by using a periodontal curette from the lingual margin to the buccal margin; **11.** Remove the excess of resin around the bracket and check with the help of a dental floss if all the contacts are free; **12.** Place the arch.

Precautions and Caution:

Exclusive use of the professional. In case of allergic reactions, suspend the use of the product. Do not store near products such as eugenol or clove oil, since they may inhibit the polymerization. Close the syringe immediately after use. Do not use the product if it is expired. Keep the product away from children. In case of accidental contact with the eyes, rinse the area with water and call a doctor immediately. When photopolymerizing, the use of a photopolymerizer with a power above 400 mW/cm² is recommended.

Storage: Store the syringe in a fresh, dry place, protected from light. Keep in a temperature range from 2° to 30° C. Do not keep near materials which contain eugenol or clove oil in its composition.

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