**Presentation:**

Flex Retractor size L / M..... 01 unit  
 Expandex Lip Retractor Adult / Children size Autoclave..... 01 unit  
 Lateral Cheek in U Adult / Child size Autoclave..... 02 units  
 Lateral Cheek in V Adult / Child size Autoclave..... 02 units

**Description:**

The remover is a medical device that assists in the opening of the lips, facilitating various procedures in dentistry. Its use is basically indicated to remove the labial mucosa and cheek for brackets bonding, prophylaxis, intraoral photographs, fluoride application and adhesive restorations, providing a greater opening of lips and cheeks. It features modern, comfortable and safe design for patient use.

**Indication:**

Assisting intraoral photographs of posterior teeth, brackets bonding, prophylaxis, fluoride application and adhesive restorations.

**Composition:**

Expandex and Flex: Polyacetal. U and V-shaped side: Polycarbonate.

**Cleaning and sterilization procedure:**

**1.** After using the product, make the removal of organic matter in ultrasonic tubs by using enzymatic soap (minimum 3 enzymes) under dilution recommended by the manufacturer. **2.** Rinse under running water and observe the presence of residues, repeat the process or make manual removal. **3.** For autoclave sterilization, after cleaning, arrange the parts in the packaging for sterilization and autoclave at 121 °C temperature (250 °F), according to the autoclave manufacturer's recommendations.  
 \* Removers have a validity of 50 cycles for sterilization.

**Directions of use:**

Directions of use- First, make sure the product is clean and sterilized;  
 Moisten the lips of the patient;  
 Position the instrument as necessary skills and training;  
 After using perform the cleaning and sterilization process;  
 Store the product in the package itself for sterilization.

**Warning and precautions:**

The use is restricted to professionals specialized in dentistry, according to local legislation and with specific training for its handling.  
 Procedures provided in the directions of use should be strictly followed.  
 Use only after the application of cleaning and sterilization procedures.  
 Use the correct size of remover for each patient.  
 Do not use the product after the expiration of the allowed cycles.  
 Keep away from the reach of children.

**Contraindications and side effects:**

Patients with sensitivity to polycarbonate or polyacetal should not use the product.

**Storage and transport:**

The product should be stored in the packaging for sterilization at room temperature, sheltered from sunlight and moisture.

**Disposal:**

The product should be disposed of, according to current legislation of its country.

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