ENGLISH

Content:

1 bottle containing 28g of powder. 1 measuring spoon. 1 bottle containing 10ml of liquid.

Indication:

1. Fixing inlays, crowns and bridges.
2. Lining cavities.
3. Temporary dental restoration.

Product Description:

Zinc Cement is suitable for cement restorations of cast metal, ceramic restorations, crowns, bridges and for use as a cement base or cavity liner.

Composition:

1. Powder: Zinc oxide, magnesium oxide, bismuth carbonate and silicon dioxide;
2. Liquid: phosphoric acid 85%, aluminium hydroxide, zinc oxide, distilled water.

Directions for use:

1. Zinc oxide can be used consistently for two different purposes: one for cementation, and one for restoration and lining. The ideal temperature for mixing should be between 21 °C and 32 °C.

Consistency For Restoration and Lining

The recommended proportion is for one large scoop for 4 drops of liquid. Divide the powder and mix as showed bellow. The mixture will reach a proper consistency for restoration when it becomes a sticky material not sticking in a spatula when touched, in a way that will not break as a lint.

Consistency For Cementing

1. The recommended proportion is for one small scoop to 3 drops of liquid. The consistency of the mixture will be suitable when it is creamy and falls off the spatula in the form of sticky beads, which spread slightly when it reaches the glass plate. This consistency of the mixture sticks easily to the spatula.

2. Place the powder on the glass plate and divide into four equal parts.

3. Divide one of them, into two of the equal parts.

4. Divide the eight portions into two equal parts.

5. Shake the liquid bottle and drip an appropriate amount of liquid, according to the thickness desired, onto the plate beside one of 1/16 measures of powder and mix this portion in the liquid for 10 seconds. **6.** Add another 1/16 portion and mix for a further 10 seconds. **7.** Add an 1/8 portion and mix for a further 10 seconds. **8.** Add more 2 equal parts and mix for 10 seconds for each portion. **9.** Add the last part and mix for 30 seconds; **10.** The ideal mixing time is 1 minute and 30 seconds. The liquid must not be exposed too long; therefore it should be mixed immediately.

Precautions:

Follow the instructions precisely. Avoid excessive dehydration of the dental cavity; carry out asepsis for the dental cavity with an appropriate germicide. Keep the cement under pressure for cementation for 4 minutes. For restoration await the initial setting of the cement for 4 minutes. For the exclusive use of dental professionals. Do not use on patients with sensitivity to the product. Keep out of reach of children.

Contra-indications:

This product should not be used as a liner for the restoration of composite resins. In this case calcium hydroxide liners are recommended.

Transport and Storage:

Store the bottle in its sealed packaging, at room temperature, in a dry location, protected from light and heat.

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