ENGLISH

Content:

1 syringe of 2g of white color calcium hydroxide light curing type and 5 applicators or

1 syringe of 2g of dentin color calcium hydroxide light curing type and 5 applicators.

Indication:

White or dentin color calcium hydroxide allow the protective lining under restorative materials, cement and other base materials such as opacifier for metals and reactional dentin.

Product Description:

White or dentin color calcium hydroxide possess important properties such as biocompatibility, antibacterial action by enzymatic inhibition and cell wall changes, anti-inflammatory action and are responsible for the alkaline phosphatase tissular enzyme activation, which induces the formation of mineralized bone tissue and therefore contributes to the repair process.

Composition:

White calcium hydroxide Calcium hydroxide (7.5%), UDMA, tetraethylene glycol dimethacrylate, Butylated Hydroxtoluene, camphorquinone, Chivacure EPD, gob 1.0 µm. -

Dentin calcium hydroxide Calcium hydroxide (7.5%), UDMA, tetraethylene glycol dimethacrylate, Butylated Hydroxtoluene, camphorquinone, Chivacure EPD, gob 1.0 µm, yellow iron oxide, red iron oxide, titanium dioxide.

Directions for use:

1. Indirect Pulp cap and reactional dentin opacifier: Wash and dry the cavity. Select the Calcium hydroxide color and apply a thin layer of maximum 0.2 mm with a spatula or brush. Clear excesses and photopolymerize for 20 seconds.

2. Metal opacifier in Prosthesis repairs: Prepare the surface of the metal and ceramic with aluminium oxide jet or diamond tips to make it rough. Brush the edges of the fractured porcelain. Wash and dry. Select the white color calcium, apply a thin layer of at most 0.2 mm with a spatula or brush, exclusively on the metal. Photopolymerize for 20 seconds.

Precautions:

Exclusive use of the dentist. Do not use this product as an intracanal medication, only as a dental liner. Discard the applicator after use. Do not use in patients with sensitivity to the product. Phenolic compounds such as eugenol (clove oil), phenol, cresol, inhibit polymerization and cannot be used in preparation as liner and disinfection. Keep out of reach of children. Preferably use Light Emitting Diodes (LED) photopolymerizer, power 500 mW/cm² to 1000 mW/cm², with wavelength of 440-480 nm with high intensity.

Transportation and Storage Conditions:

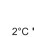
Store the syringes in a dry, cool place protected from dust, avoiding the heat, sunlight and other sources of direct light. Do not store them near materials that have in their composition eugenol or clove oil. Seal the syringes immediately after use in order to prevent exposure of the material to light. Storage temperature of 2 °C to 28 °C.

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