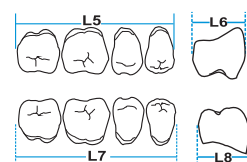
M 2 (L5)29,2 (L6)9 (L7)31,3 (L8)7,9

Posteriores, Superiores e Inferiores
Upper and Lower posteriors

M 3 (L5)29 (L6)8,9 (L7)31,1 (L8)8,0

M 4 (L5)30,3 (L6)9,5 (L7)33,4 (L8)9,1

M 5 (L5)30,7 (L6)9,2 (L7)33,4 (L8)8,5

M 6 (L5)33,2 (L6)9,9 (L7)35,4 (L8)9,3

TRILUX

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SYNTHETIC POLYMER TEETH FOR DENTAL PROSTHESES

Follow the usual procedures for choice, waxing-up, articulation, polymerization, infection-control and maintenance of the prosthesis, as well as for individual protection.

For an optimized bonding to the denture base resin, grind the tooth base with a grinding instrument. After insulating the flask and prior to pack the acrylic dough, clean the base surface of the tooth using methylmethacrylate monomer.

Indications for the patient: To increase the teeth durability it is highly recommended to avoid contact with alkalis, acids or other substances that can damage acrylic materials during the daily hygiene.

Composition: Polymethylmethacrylate - Dimethacrylate - Fluorescent - Biocompatible Pigments - Organically Modified Ceramic. **Storage:** Keep protected from light and heat.

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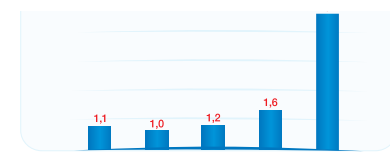
TRILUX, the first artificial tooth with improve wear resistance, water sorption and solubility in water.

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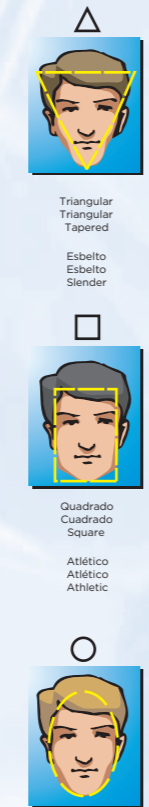
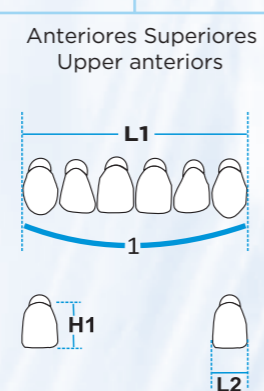
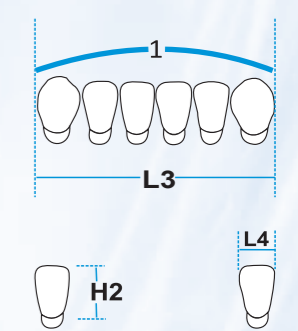
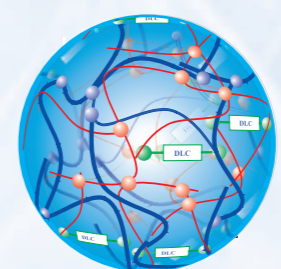
TRILUX teeth are manufactured with double cross linked DLC and high molecular weight Methacrylate Polymer, within the most modern techniques and under rigid quality control. All production processes comply with ISO 13485 and Good Manufacturing Practices (GMP) norms. They are certified Biocompatible in accordance to ISO 10993-5 and comply ISO 22112 - TYPE 1/TIPO 2.

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