d) When the instrumentation is finished, irrigate with physiologic serum.

II. IN THE ASEPSIS OF THE CAVITARY PREPARE:

a) With a cotton ball, clean the cavity with a small amount of **CLOREXORAL GEL 2%**;

b) Remove the excess **CLOREXORAL GEL 2%** and dry the cavity with airflow. The use of water in the cavity is optional. Dregs of **CLOREXORAL GEL 2%** do not interfere in the adhesion of the material.

SPECIAL CARES:

Do not ingest.

Do not use as an oral rinse.

Avoid needless contact with tongue and other parts of the mouth.

CLOREXORAL GEL 2% has an extremely bitter taste and when in contact with the tongue may cause a temporally taste change and heat.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

CLOREXORAL GEL 2% should not be used in patients known to have sensibility to any of the ingredients listed.

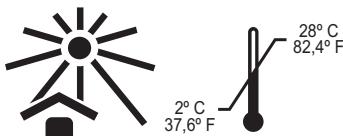
WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions. In case of accidental exposure to oral mucosa, slight local irritation may occur. Remove and rinse with water.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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Reg. ANVISA:10298550089.

INDICATION:

CLOREXORAL GEL 2% It is used for root canal irrigation, to help in the endodontic instrumentation and to asepsis the cavity prepare.

COMPOSITION:

CLOREXORAL Gel 2%: Chlorhexidine Digluconate (2%), Methylparaben, Hydroxyethylcellulose and Deionized Water.

TECHNICAL INFORMATION:

The microorganisms and theirs metabolic products are considered the responsible for pulpal and periapical pathologies. Because of this, the complete cleaning of the canal root systems is essential to the endodontic treatment success. The principle active of **CLOREXORAL GEL 2%** is Chlorhexidine Digluconate. The chlorhexidine was introduced, more than 30 years ago, as a general disinfectant for gram-positive and gram-negative bacteria. Chlorhexidine when in low concentration (0,12%) is a bacteriostatic agent and in high concentration (2%) become in a fast bactericide agent.

Chlorhexidine act through an electrostatic interaction of its positives and negatives charges in the bacteria's cellular wall, and braking it, precipitating the cytoplasm and causing the bacteria's death. It has substantivity and its residual effect is about 48 hours. It has low toxicity do not harming the periapical tissue.

USE INSTRUCTIONS:

I. AS AUXILIARY IN THE ENDODONTIC INSTRUMENTATION:

- Insert the product in the canal, until it is totally full;
- Realize the instrumentation and when the saturation occurs, irrigate with physiologic serum;
- Repeat this procedure as many times as needed;