# Dentflex

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*The Air Scaler is one dental unit accessory. It is easy to handle this equipment, it may be coupled in any kind of unit, by following all the products directions for its adequate use.*

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*The equipment is used in dental treatment, dental prophylaxis, plaque removal and calculus above and subgingival, there is removal and reduction of biofilm mineralized and non-mineralized.*

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*The Air Scaler is designated for dental procedures and can only be used and installed by a qualified and capacitated professional on dental area. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.*

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*The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.*

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*Do not clean the equipment using ultra sonic cleaners*

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*The equipment must be sterilized before and after each treatment. Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.*

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*Sterilize the equipment before using it for the first time. Use compressed dry air, filtered and with no oil.*

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*Keep in a place protected from the sun, heat, dust and humidity;  
Keep away from chemicals and cleaning agents;  
Carry out the transport only in its respective packaging;  
Ambient temperature (transport and storage): -10°C to 55°C;  
Relative air humidity (transport and storage): 0% to 85%;  
Atmospheric pressure: 70kPa to 106kPa;  
Maximum stacking: 8 units.*

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1 - Fit the Air Scaler to the unit's terminal observing the ducts direction.

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2 - Screw the unit's terminal extremity to get fixed to the equipment.

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1- The dental unit's terminal must be manufactured in accordance with ISO 9168 norm.

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Universal tip, suitable for accomplishing works on all surfaces of teeth.

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The tips Perio SUB and Perio E are optional and do not come with the product.

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
Tip suitable for deep subgingival works on pins and cements removal, etc.

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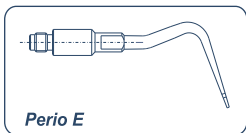*Suitable for work on vestibular, lingual or cervical faces of the anterior teeth.*

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
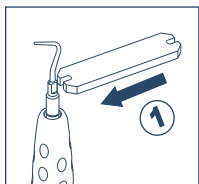
*2 - To release the tip, hold the equipment and turn the key counter-clockwise (fig. 7).*

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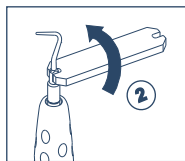
3 - Now to fix the tip, hold the equipment and turn the key clockwise (fig. 8).

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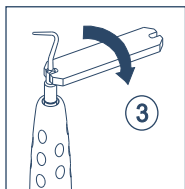*Like any other electromedical device, Cavflex 6000 is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).*

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*Dent-Flex Ind. E Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in Cavflex 6000 have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: distilled water and tip.*

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*All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.*

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*SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS*

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*The Cavflex 6000 was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.*

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*For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:*

- *Installation and/or operation in disagreement with this manual;*
- *Fall;*
- *Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause.*

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*With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.*

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

*To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: [sac@dentflex.com.br](mailto:sac@dentflex.com.br) / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.*

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