DESCRIPTION:

Power Bleaching is a whitening gel for home use based on carbamide peroxide, in concentrations of 10%, 16% or 22%. Used for whitening vital and non-vital teeth with scientific technology to provide the desired result with maximum comfort during treatment. The whitening effect of the product can already be observed in the first hours of treatment. Its formulation and the protocol of use were developed to provide reduced sensitivity resulting from the treatment, reduced time of daily application, adaptation to the patient's routine and easy handling, all this with the aim of recovering the aesthetics of the teeth from an excellent whitening.

INDICATION:

Power Bleaching is intended for tooth whitening in the following cases: **Vital teeth** - naturally dark or yellowed, darkened by dyes, age, trauma, stained to a mild degree by tetracycline and by fluorosis. **Non-vital teeth** - in association with internal tooth whitening.

COMPOSITION:

Carbamide Peroxide, Potassium Oxalate, Sodium Fluoride, thickener, neutralizer, preservative, humectant, and purified water.

MODE OF USE:

1. Brush your teeth properly; 2. Test the trays in position several times; 3. Attach the tip to the syringe, detach the cap from the tip, and place 1 drop of the product in the tray on the locations of the teeth to be whitened; 4. Take the trays one by one into your mouth and press them with your thumb and forefinger against your teeth to spread the gel over the entire area to be whitened; 5. Remove the excess gel with the help of your index finger; 6. After using the tray loaded with gel for the recommended time (from half an hour to two hours daily for the 10% - 16% concentration and from half an hour to one hour daily for the 22% concentration), remove the tray; 7. Clean the tray with the help of the toothbrush and running water, dry and store the tray in the tray holder; 8. Rinse your mouth with water and, if necessary, brush your teeth again.

STORAGE AND TRANSPORT:

Keep the syringe tightly closed, at room temperature up to 25 °C, humidity below 75 UR, protected from light and heat. After opened keep syringe in refrigerator.

WARNINGS:

Power Bleaching, has been developed to be self-administered by the patient, should be used with the supervision of a dentist. However, in the presence of severe sensitivity, reduce the number of hours of bleaching per day, reduce the amount of bleaching gel used in the tray and, if necessary, discontinue bleaching for a few days and consult your dentist again. The dentist must inform the patient about situations in which there may be failure or partial success of the treatment, considerable color differences between the teeth, presence of restorative materials and prosthetic restorations, among others. Sold under prescription.

PRECAUTIONS:

Product should be kept out of reach of children. If whitening gel comes into contact with the eyes or skin, rinse under running water and seek advice from your dentist if symptoms persist. In case of ingestion of a large amount of the whitening gel, drink plenty of water and seek medical advice. Do not eat, drink or smoke while wearing the tray filled with whitening gel. Try to avoid, during the days you are whitening your teeth, the ingestion of beverages and foods with dyes. If you are going to consume them, do it with moderation. In case of tooth sensitivity or gum burning, stop the treatment and contact your dentist. Use the gel at room temperature. Do not reuse the product or packaging and dispose of in accordance with local legislation.

CONTRAINDICATIONS:

Do not use the product in patients with hypersensitivity to the components of the formula. The product should not be used for internal tooth whitening.

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