Presentations:**A2 Kit:** 1x10g A2 powder vial, 1 x 8g liquid vial, 1 dosing spoon, 1 mixing pad;**A3 Kit:** 1x10g A3 powder vial, 1 x 8g liquid vial, 1 dosing spoon, 1 mixing pad.

Indication: Can be used in the ART (atraumatic restorative treatment) technique, in all classes of restorations of deciduous teeth, provisional restorations of permanent teeth (Class I, II, III and IV), as a base for final restoration, non-carious lesions and provisional cementation of prosthetic parts. Ionglass R has good adhesion to enamel and dentin, rendering the creation of retention unnecessary.

Product description: Ionglass R is a highly resistant self-curing glass ionomer, aesthetic, with viscosity suitable for restorative work. Has excellent biocompatibility and low risk of postoperative sensitivity. Have a great marginal sealing in the cavity and prevents the formation of cracks.

Composition: Powder: Sodium fluorosilicate, Calcium and Aluminum.
Liquid: Polyacrylic acid, Tartaric acid and Purified water.

How to use: **1.** Remove the carious tissue, without the need to make retentions. In very deep cavities only, it is recommended to use a calcium-hydroxide based liner. **2.** Prepare the material as follows: 1 flat dosing spoon of powder to 1 drop of liquid. Using the mixing pad or a glass plate, place one measure of powder and one drop of liquid, and add the powder to the liquid gradually using the plastic spatula for ionomers, Ionomix® (Maquira). Take care to ensure that the mixing time does not exceed 01 minute. **3.** Apply the material inside the cavity straight away. During the setting phase avoid contact with saliva and water, as the material is vulnerable to dissolving on its outer surface. To protect the restoration from moisture, coat with a layer of varnish. **4.** The setting time of the material is 2 to 3 minutes. After this time the excess can be removed with appropriate tools. **5.** For a better finish and polish, wait until 24 hours after the insertion of the material. **6.** After use rinse the materials used with cold water before the material residues solidify.

Note: The temperature range for the times specified is 18°C to 32°C. Temperatures above this range decrease the working time and temperatures below this range increase the time specified.

Precautions: Follow the instructions for use strictly; If the region to be treated has a pulp exposure, lining is recommended using a Calcium Hydroxide such as PA® or Hidrox-Cal®, for example; The bond strength of this material diminishes upon contact with eugenol, phenol or paramonochlorophenol, therefore it should not be used in conjunction with these medicines; Exclusively for use by dental professionals; Do not use on patients with sensitivity to the product; Keep out of reach of children; The direct application of the product onto the dental pulp may cause irritation or irreversible damage (death of pulp).

Transport and Storage: Store the vial tightly closed at room temperature in a dry place, protected from light and heat.

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