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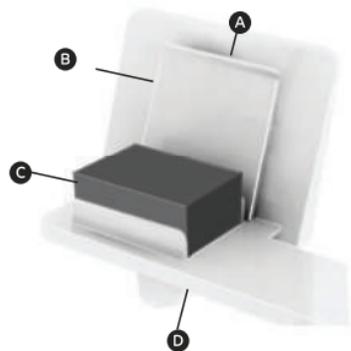
EN - Put the X-Ray film firmly in the right position, between the support (for film) and the pressure device as shown in the illustration.

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EN - X-ray positioner: In order to place the piece in the mouth of the patient, the film and the tooth must be in parallel. During the positioning of the upper and posterior teeth, the arc must touch the front side of the patient. It is noticed that in an x-ray of posterior teeth, the bite is always in the area of the anterior teeth. When you want to dissociate overlaid roots, mesialize or distalize the frame in the mouth, approximately 20 degrees horizontally, and then there will be a mesialization or distalization without distortion.

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EN - Projection: The constituents of this KIT must be used according illustrated image. Where the angle of X-ray is properly combined with the position of the film and the tube in parallel to the positioner.

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ENGLISH - STERILIZATION

After use, the pieces of the kit must be sterilized within specific packages, such as sterilization pouches, at 121° for 30 minutes.

Do not put the pot and its lid of the kit into the autoclave.

Remember: Take the bite device off the positionator to sterilization.

The sterilization is valid for one week if not packaged. It can also be disinfected using chemical disinfection, for example, a solution of 0,2% acid peracetic for 1 hour. Then rinse the parts, dry and store in covered jar.

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