PT

Isolar

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EN

Isolar

Indication: Indicated for insulation of plaster casts in prosthetic procedures using acrylic resins.

Use: Shake the bottle before use. Hydrate the plaster model for 10 minutes to avoid absorption of Isolar before its application. Pour the liquid into a pot (Dappen). Using a clean brush, apply Isolar on the entire surface of the plaster cast to be insulated. It is recommended the application using large brush strokes, always in the same direction. Isolar should be continually applied until a smooth and shiny texture is achieved. Wait for 5 minutes and proceed with the application of the acrylic resin on the model. In models featuring teeth, insulate mainly the retentive areas, avoiding application to the posterior parts of the teeth. In cases where a film is used (cellophane or plastic sheets) as insulation during the first pressing, Isolar must only be applied to the model after the film's removal.

Warnings, precautions and restrictions:

SHAKE THE BOTTLE BEFORE USE. For professional dental and prosthetic use only. Product is not flammable. Non-toxic and non-volatile product. Keep the packaging always closed. If not followed the instructions of use it can affect the performance of the product. Do not return the poured liquid to its respective pot, thereby avoiding contamination of the bottle with plaster residue. Avoid contact with skin and eyes. In case of contact with skin: Wash with plenty of water. In case of contact with eyes: Wash with plenty of water for 15 minutes. In case of accidental ingestion: consult a doctor.

Composition: Sodium alginate, preservatives and water.

Storage: To be stored in dry and fresh place, protected from sunlight.

Disposal: Waste must be disposed of in an industrial or sanitary landfill, in compliance with local legislation in force. Never dispose of waste in sewers, waterways or household waste.

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