Palaseal[®]

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GB Instructions for use

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Stand: 2021-06

GB Instructions for use Palaseal[®]

Medical Device – for use by dental health care professionals only.

Composition
(2,4,6-trioxo-1,3,5-triazinane-1,3,5-triyl)triethylene triacrylate; methyl methacrylate; diphenyl(2,4,6-trimethylbenzoyl)phosphine oxide; Bisphenol A Diglycidylether Diacrylate (BADGE-DA); glycerol; propoxylated; esters with acrylic acid

Intended purpose
This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant. KULZER medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.

Light curing lacquer for surface sealing of dentures, temporary crowns and bridges made of PMMA acrylic. With Palaseal it is possible to achieve a smooth surface on newly finished and reworked PMMA-based prosthetic components. The transparent lacquer does not alter the benefits of the prosthetic component. The smooth surface prevents mechanical irritations of the tongue and mucous membrane.

Indications
Surface sealing of PMMA-based dentures, temporary crowns & bridges.

Contraindications
In case of known or suspected allergy to components of the product, its use is contraindicated. Do not use in case of known or suspected allergy against (meth)acrylate compounds or benzoyl peroxide.

Side effects
Hypersensitivities to the product or its components cannot be excluded in individual cases. In case of suspicion please contact the manufacturer for ingredients information. Hypersensitivity reactions (allergies), local sensory disturbances, taste disturbances and irritation of the oral mucosa have been described in isolated cases. After processing, all MMA/PMMA-based denture parts that always contain a permitted, small proportion of residual monomer (<4.5%). For further risk reduction of intolerance reactions, the denture has to be stored in tepid water for at least 12 hours before placement.

Processing
Smooth prosthesis with emery paper then polish with pumice stone. Do not use any oil-based cleaning agents. Blow dry surface well using oil free air. Use a soft disposable brush to apply a thin, even layer in one direction without rebrushing. Avoid air pockets. After an exposure time of approx. 20 sec, carry out polymerisation for 90 sec in the HiLite[®] power, Heraflash[®]. Position the prosthetic in the light curing unit so that the area coated with Palaseal is irradiated completely. Repeat polymerisation a second time (90 sec irradiation time) if there are any shaded areas (vestibular fold upper/lower jaw, sublingual areas and interdental areas). In this case, arrange the prosthetic so that the shaded areas are irradiated completely. Refinishing is not necessary as a smooth, cured surface is achieved.

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Storage advice
Do not store above 25°C (77°F). Do not use after expiry date. Keep out of direct sunlight. Reseal carefully after removing product. Store out of reach of children. Visually inspect for damage before first use. Damaged products must not be used.

Warnings and safety instructions
May cause skin and eye irritation. Avoid contact with skin. Skin contact may cause sensitization. Wear suitable protective gloves. In case of contact with skin, immediately wash with plenty of water and soap. Avoid contact with eyes. In case of contact with eyes, rinse with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If symptoms persist, consult an eye specialist. Highly flammable liquid and vapour. Flash point: 13°C. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources – no smoking. May cause respiratory irritation. Avoid breathing dust/fume/gas/mist/vapours/spray. Toxic to aquatic life with long lasting effects. Suspected of damaging fertility.

When giving feedback on the product, please always state batch designation and article number. According to EU Medical Device Regulation users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

Disposal information
Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

Safety data sheets and more information are available at our website **www.kulzer.com**

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® = registered trademark of Kulzer GmbH
® = marca registrada de Kulzer GmbH
® = marca registrada da Kulzer GmbH